
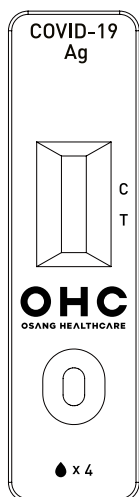


OHC GeneFinder™ COVID-19 Ag Rapid Test

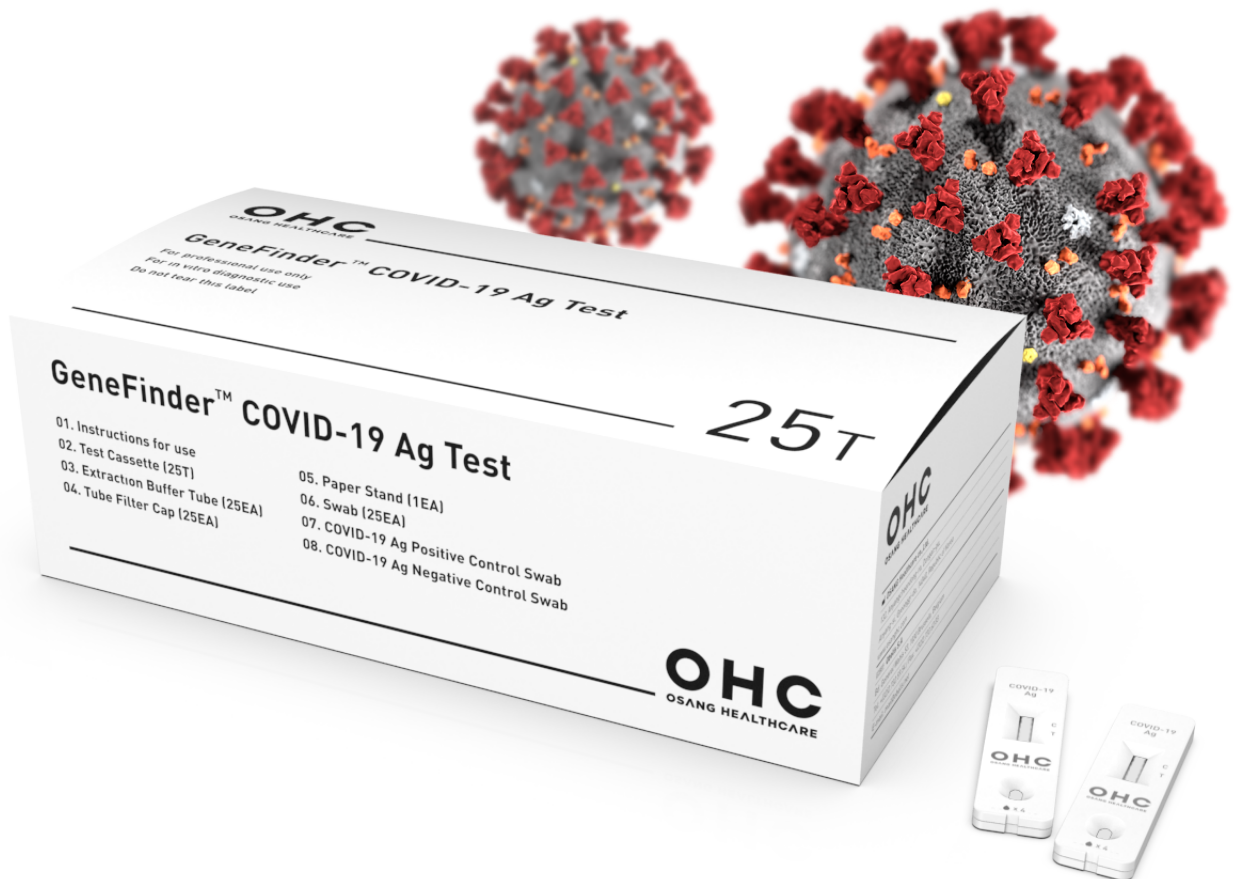

STRONG POINTS

- Suitable and simple detection of SARS-CoV-2 antigen
- User friendly Cassette Type & Color
- Easy to use
- Room temperature storage
- Fast result within 15 min
- Point-Of-Care Diagnostic Test


PROFILE

Intended use	Detection of SARS-CoV2 antigen
Time to Result	Within 15 minutes
Specimen	Nasopharyngeal Swab
Storage	2-30°C / 36-86°F
Shelf Life	18 Months
Pack Size	25 Tests / Box


COMPONENTS

- Instructions for use
- 25 GeneFinder™ COVID-19 Ag Rapid Test cassettes
- 25 Extraction buffer tubes
- 25 Tube filter cap
- Sterilized nasopharyngeal swabs for sample collection
- 1 Positive control swab
- 1 Negative control swab


PROCEDURE

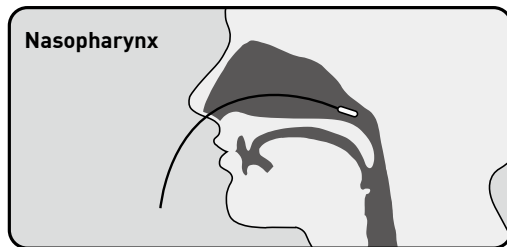

Insert a sterile swab into nasopharynx, and collect the specimens.

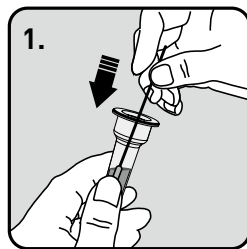


Stir the sample swab in the extraction buffer tube more than 10 times.

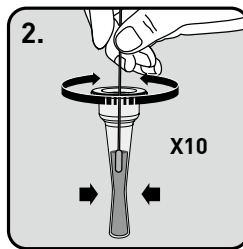


Squeeze out the swab by squeezing the extraction buffer tube with your fingers to extract all sample collected.




Remove and discard swab.

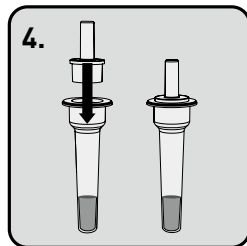

Place the nozzle cap with tube.





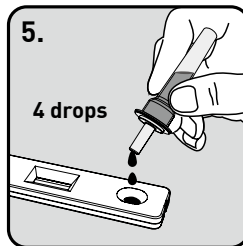


Apply 4 drops (~100 uL) of the solution into the sample well.

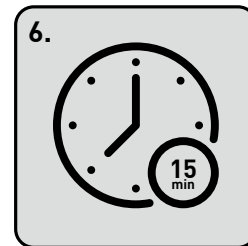

Read after 15 mins.

INTERPRETING TEST RESULTS

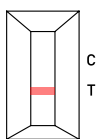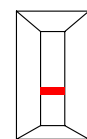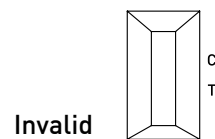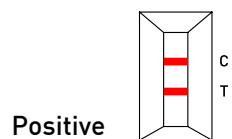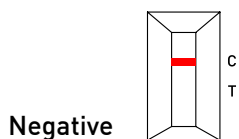

CLINICAL PERFORMANCE


The performance of the GeneFinder™ COVID-19 Ag Rapid Test was established with 100 nasopharyngeal swabs prospectively collected and enrolled from individual patients who tested with a RT-PCR method for SARS-CoV-2 infection. All specimens within a pre-specified date range were selected and then sequentially tested in a blinded fashion. The performance of the GeneFinder™ COVID-19 Ag Rapid Test compared to RT-PCR for nasopharyngeal swabs.

		RT-PCR		
		Positive	Negative	Total
GeneFinder™ COVID-19 Ag Rapid Test	Positive	29	1	30
	Negative	1	99	100
	Total	30	100	130

- Sensitivity: 96.7% [95% CI: 83.3% - 99.4%]

- Specificity: 99.0% [95% CI: 94.6% - 99.8%]

HOW TO USE

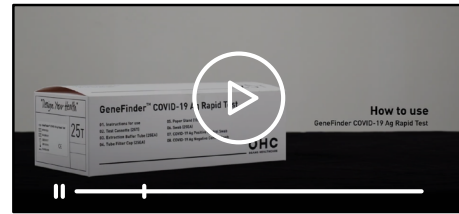

Search for "GeneFinder COVID-19 Ag Rapid Test"